

PICC Fill Volumes

Remember - this is just the fill volume of the line untrimmed without a bionector

Include bionector volumes

Flush with a larger volume to pass the end of the line to prevent occlusion

PICC	Fr	Double Lumen	CM	ML
Angiodynamics BioFlo	3Fr		55cm	0.8ml
Angiodynamics BioFlo	4Fr		55cm	1.1ml
Angiodynamics BioFlo	5Fr	Yes	55cm	1.2mls
Bard Power PICC SV	4Fr	Yes		Red - 0.51mls / White - 0.38mls
Cook	3Fr		50cm	0.5ml
Cook	4Fr		60cm	0.6 ml
Cook	5Fr		60cm	0.8ml
Vygon Premicath	1Fr			0.09ml
Vygon	2Fr		15cm	0.12mls
Vygon	2Fr		30cm	0.12mls
Epicutaneo	2Fr		30cm	0.12ml
Vygon	3Fr		30cm	0.16ml
Vygon	3Fr	Yes	6cm	0.09ml
Vygon	3Fr	Yes	10cm	0.09ml
Vygon	3Fr	Yes	20cm	0.19ml
Vygon	4.5Fr	Yes	60cm	0.5 ml
Nutriline Twinflo	2Fr	Yes		0.2ml
Leaderflex	2Fr		20cm	0.15ml
Bionector MicroClave				0.04mls
Bionector TKO				0.07mls